

Detergent for Histareader • Instructions for use •



REF: RLA705

Intended purpose & product description

The device is a consumable, which is used for performing the basophil histamine release assay (BHRA).

The detergent for Histareader is an aqueous solution of potassium hydroxide (< 0.1 %) and is included when the 'full rinse' cycle in the Histareader has been performed.

For in vitro diagnostic use.

Intended user

The detergent is for professional (health care) use only.

Composition

Contains potassium hydroxide < 0.1 %

Package size

| Product name | Article no | UDI code | Volume | Pack size |
|---------------------------|------------|----------------|--------|-----------|
| Detergent for Histareader | RLA705 | 05744003280094 | 100 mL | 1 |

Material & equipment required but not provided

Histareader rinse tray


The Histareader 'full rinse' also requires: -Distilled H₂O
-Acidic clean for Histareader

Full rinse procedure


Histareader full rinse procedure

'Full Rinse' is performed after daily use of the Histareader and it is also recommended initially, when the Histareader has not been used for a longer period.


1) Add approx. 5 mL detergent into the rinse tray:

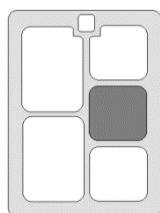

2) Add approx. 5 mL acidic clean into the rinse tray:

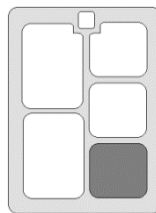

3) Add distilled H₂O into the rinse tray. Approx. 10 mL in the large section and 5 mL in the smaller:

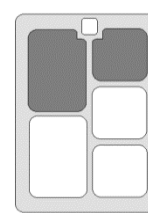

Detergent for Histareader • Instructions for use •



REF: RLA705

Storage and handling conditions

Store at +2 - +8 °C until labelled expiry.

Warnings

The detergent for Histareader is a non-hazardous reagent








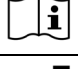

Waste disposal

Non-hazardous waste

Manufacturer

RefLab ApS
Copenhagen Bioscience Park 'COBIS'
Ole Maaloes Vej 3
DK 2200 Copenhagen N
Denmark

Symbols

| | |
|---|------------------------------|
|  | CE mark |
|  | For in vitro diagnostic use |
|  | Catalog number |
|  | Batch code |
|  | GTIN code |
|  | Expiry date |
|  | Storage temperature |
|  | Consult instructions for use |
|  | Manufacturer |